



## Pacific Blue™ anti-human CD123

Analyte Specific Reagent. Analytical and performance characteristics are not established.

Catalog# / Size 983708 / 500 μL

Clone 6H6

Other Names IL-3Rα, IL-3 Receptor alpha

**Isotype** Mouse IgG1, κ

Description CD123 is the 70 kD transmembrane α chain of the IL-3 receptor. Alone, CD123 binds IL-3 with

low affinity; when CD123 associates with CD131 (common  $\beta$  chain), it binds IL-3 with high affinity. CD123 does not transduce intracellular signals upon binding IL-3 and requires the  $\beta$  chain for this function. CD123 is expressed by myeloid precursors, macrophages, dendritic

cells, mast cells, basophils, megakaryocytes, and some B cells.

## **Product Details**

Reactivity Human

Formulation Phosphate-buffered solution, pH7.2, 0.09% sodium azide, 0.2% (wv) BSA (origin USA), and a

stabilizer.

Preparation The antibody was purified by affinity chromatography and conjugated with Pacific Blue™ under

optimal conditions.

Concentration 50 µg/mL

Storage & Handling The antibody solution should be stored undiluted between 2°C and 8°C, and protected from

prolonged exposure to light. Do not freeze.

Application Suggested for Flow Cytometry

**Disclaimer** WARNINGS AND PRECAUTIONS

 Use appropriate personal protective equipment and safety practices per universal precautions when working with this reagent. Refer to the reagent safety data sheet.

This antibody contains sodium azide. Follow federal, state and local regulations to dispose of this reagent. Sodium azide build-up in metal wastepipes may lead to explosive conditions; if disposing of reagent down wastepipes, flush with water after disposal.

All specimens, samples and any material coming in contact with them should be considered potentially infectious and should be disposed of with proper precautions and in accordance with federal, state and local regulations.


4. Do not use this reagent beyond the expiration date stated on the label.

5. Do not use this reagent if it appears cloudy or if there is any change in the appearance of the reagent as these may be an indication of possible deterioration.

6. Avoid prolonged exposure of the reagent or stained cells to light.

Alexa Fluor® and Pacific Blue™ are trademarks of Life Technologies Corporation.

View full statement regarding label licenses

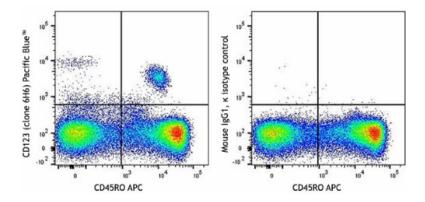

Typical results from Human peripheral blood lymphocytes stained either with 6H6 Pacific Blue™ used at 5 µL/test (left) or with an isotype control (right), with anti-human CD45RO APC costaining.

## Symbols Glossary\*

Symbol	Meaning	Symbol Title	Symbol No.	Symbol	Meaning	Symbol Title	Symbol No.
REF	Catalog number	Catalogue number	5.1.6	(i	Indicates the need for the user to consult the instructions for use.	Consult instructions for use	5.4.3
1	Indicates the temperature limits to which the medical device can be safely exposed.	Temperature limit	5.3.7	淡	Indicates a medical device that needs protection from light sources.	Keep away from sunlight	5.3.2
K	Indicates the upper limit of temperature to which the medical device can be safely exposed.	temperature	5.3.6	Ω	Indicates the date after which the medical device is not to be used.	Use-by date	5.1.4
•••	Indicates the medical device manufacturer.	Manufacturer	5.1.1	EC REP	Indicates the authorized representative in the European Community.	Authorized representative in the European Community	5.1.2
	Indicates the manufacturer's	Batch code	5.1.5		Indicates a medical device that is	In vitro diagnostic	5.5.1
LOT	batch code so that the batch or lot can be identified.			IVD	intended to be used as an in vitro diagnostic medical device.	medical device	

<sup>\*</sup> Symbol information is from EN ISO 15223-1:2016 Medical devices – Symbols to be used with medical device labels, labelling and information to be supplied – Part 1: General requirements

Not for resale. BioLegend will not be held responsible for patent infringement or other violations that may occur with the use of our products.

\*These products may be covered by one or more Limited Use Label Licenses (see the BioLegend Catalog or our website, www.biolegend.com/ordering#license). BioLegend products may not be transferred to third parties, resold, modified for resale, or used to manufacture commercial products, reverse engineer functionally similar materials, or to provide a service to third parties without written approval of BioLegend. By use of these products you accept the terms and conditions of all applicable Limited Use Label Licenses.

8999 BioLegend Way, San Diego, CA 92121 www.biolegend.com Toll-Free Phone: 1-877-Bio-Legend (246-5343) Phone: (858) 768-5800 Fax: (877) 455-9587